

Panther MP 627 GP



Post-Op Cold Therapy

The Panther MP 627 GP is a Panther MP 627 brace with a hot-cold gel pack attached. Postoperatively, the gel pack provides cold therapy to reduce swelling.

When cold therapy is no longer needed, the patient may use the Panther LSO for relief of occasional pain.

The Panther MP 627 is universal. No tools are needed to size it. Just add or remove belt segments.

PDAC Approved: L0627/L0642



SCAN FOR
PANTHER CATALOG



The Best Value in Spinal Bracing

800-867-6070 | www.bluedortho.com

Panther MP 627 CP



SPECIFICATIONS

- Gel Pack Holder
- Hot/Cold Gel Pack
- Bilateral pull handles
- Breathable-mesh belt
- Low-profile posterior plate
- Conforms to any body shape
- Smooth dual-pull compression

UNIVERSAL SIZE

- One size fits all. 25" to 66" and beyond! Just add or remove belt segments.

PDAC APPROVED

- L0627 Custom Fit
- L0642 Off the Shelf (OTS)

LORDOSIS

The posterior polymer plate is preset at 15°. It may be heat molded for exact fit during custom fitting.



Cold Therapy Application

Freezer

- Place room-temperature Gel Pack in freezer for 1 to 2 hours.
- For best results, place Gel Pack flat in freezer.
- If you prefer, you may leave Gel Pack in freezer until needed.
- Insert frozen Gel Pack inside Gel Pack Holder, or wrap in towel, and place on area of body to be treated.



Hot/Cold Therapy Gel Pack Instructions for Use



Heat Therapy Application

Microwave

- Heat Gel Pack at full power for 50 seconds. (Time is based on 1000-watt microwave).
- Remove Gel Pack from microwave. If not hot enough, return Gel Pack to microwave for 20 seconds. Repeat as needed, but do not exceed an additional 60 seconds.
- Insert heated Gel Pack inside Gel Pack Holder, or wrap in towel, and place on area of body to be treated.

Caution

- Consult your physician before using this product.
- Heat should NOT be used if patient has circulatory problems.
- Heat Gel Pack in microwaveable-safe container.
- If heat is uneven, gently knead Gel Pack.
- Wait one hour after heating before re-heating.
- Over-heating could cause skin burns or rupture Gel Pack.
- Whether hot or cold, place Gel Pack on area of body to be treated for 15-20 minutes, or as medically prescribed.
- A child should be supervised by parent or guardian when using Gel Pack.
- Do not step on or abuse Gel Pack. Excessive force may cause rupture or leakage.
- Do not use sharp utensils to handle Gel Pack.
- Discard Gel Pack if leakage occurs.

Blue Diamond Orthopedic | 800-867-6070 | www.bluedortho.com



Blue Diamond Orthopedic | 6439 Milner Blvd, Suite 4, Orlando, FL 32809 | 800-867-6070 | 407-613-2001 | bluedortho.com

© 2021 Blue Diamond Orthopedic. All rights reserved. Patent Pending

20220422